

Retinal Camera Maintenance



Cleaning and Disinfecting Instructions

The following instructions are intended for the cleaning of the external surfaces of Topcon retinal cameras and the disinfection of the parts of the instrument that come in contact with the patient:

- Forehead rest
- Body cover
- Chinrest bars
- Touch screen



WARNING!

Be careful not to get the retinal camera internal parts wet and clean it only as indicated to prevent damaging it. Never use solvents or other abrasive agents.

Retinal cameras come with a dust cover to be used to protect them. Cover the retinal camera if it is not going to be used for a long period of time.



WARNING!

Before turning on the retinal camera, remove the cover. Never put the cover on when the illumination is on.

CLEANING THE BODY OF THE RETINAL CAMERA

Regularly clean dust off the device using a soft cloth. For more persistent superficial dirt, use a soft cloth dampened with water or alcohol at maximum 75%. Never use solvents or a chemical duster.



WARNING!

Use lint-free cleaning cloths when wiping off the lenses.

CLEANING AND DISINFECTING THE FOREHEAD REST, BODY COVER AND CHIN REST

For daily cleaning

Wipe the forehead rest and the chin rest with a cloth moistened with a warm solution of neutral detergent.

For disinfection

Option 1 - Wipe the forehead rest, chin rest side bars and chin rest cup with a cloth moistened with 75% (maximum) alcohol and let air dry.

Option 2 - Wipe the forehead rest, chin rest side bars and chin rest cup with a cloth moistened with a diluted solution of hypochlorous acid water and let it air dry.