

Slit Lamp Maintenance



Cleaning and Disinfecting Instructions

The following instructions are intended for the cleaning of the external surfaces of Topcon slit lamps and the disinfection of the parts of the instrument that come in contact with the patient.

- Forehead rest
- Slit lamp body
- Chinrest bars
- Chinrest cup



WARNING!

Be careful not to get the slit lamp illumination unit wet and clean it only as indicated to prevent damaging it. Never use solvents or other abrasive agents.

Slit lamps come with a dust cover to be used to protect them. Cover the slit lamp if it is not going to be used for a long period of time.



WARNING!

Before turning on the slit lamp, remove the cover. Never put the cover on when the illumination is on.

CLEANING THE BODY OF THE SLIT LAMP

Regularly clean dust off the device using a soft cloth. For more persistent superficial dirt, use a soft cloth dampened with water or alcohol at maximum 75%. Never use solvents or a chemical duster.



WARNING!

Use lint-free cleaning cloths when wiping off the lenses and mirror.

CLEANING AND DISINFECTING THE FOREHEAD REST, CHINREST BARS AND CHIN REST

For daily cleaning

Wipe the forehead rest and the chin rest with a cloth moistened with a warm solution of neutral detergent.

For disinfection

Option 1 - Wipe the forehead rest, chin rest side bars and chin rest cup with a cloth moistened with 75% (maximum) alcohol and let air dry.

Option 2 - Wipe the forehead rest, chin rest side bars and chin rest cup with a cloth moistened with a diluted solution of hypochlorous acid water and let it air dry.